

Viewer global keyboard shortcuts:

Shortcut	Function
F1	Help*
F11	Full screen
Ctrl + F	Forward
Ctrl + E	Export

2D viewports keyboard shortcuts:

Shortcut	Function
O	Reference Lines*
L	Auto or manual link translation synchronization*
U	Unlink translation synchronization*
Escape	Display Reset*
C	Cine Tools*
Z	Select image zooming tool*
Ctrl + Mouse wheel	Zoom
T	Select image panning tool*
Shift + T	Align to left corner
Shift + U	Align to right corner
Shift + Y	Align to center

Ctrl + Left click + Drag	Pan
W	Select image window (brightness/contrast) adjustment tool*
D	Select distance measurement tool*
A	Select angle measurement tool*
Ctrl + Del	Delete all measurements from active viewport
Del	Delete selected measurement from active viewport
R	Select image rotate tool
Shift + R	Rotate Right
Shift + L	Rotate Left
Shift + V	Flip vertical
Shift + H	Flip horizontal
Shift + A	Axial reconstruction
Shift + C	Coronal reconstruction
Shift + S	Sagittal reconstruction
Shift + M	Multiplanar reconstruction
Shift + Mouse wheel	Rotate image with 5 degree steps
Shift + Del	Clear transform
M	Select magnifier tool
Mouse wheel	If magnifier tools is active will change magnifier zoom

Alt + Mouse wheel	If magnifier tools is active will change magnifier dimensions
Ctrl + Num 0	Fit image to screen
Ctrl + Num 1	Set zoom to 100% (1 image pixel = 1 screen pixel)
Ctrl + Num 2	Set zoom to 200%
Ctrl + Num 3	Set zoom to 300%
Ctrl + Num 4	Set zoom to 400%
Ctrl + +	Increase zoom
Ctrl + -	Decrease zoom
Num 0	Full dynamic brightness/contrast
Num 2	Windowing preset Abdominal for CT images (wl=60, ww=400)
Num 3	Windowing preset Angio for CT images (wl=300, ww=600)
Num 4	Windowing preset Bone for CT images (wl=300, ww=1500)
Num 5	Windowing preset Brain for CT images (wl=40, ww=80)
Num 6	Windowing preset Chest for CT images (wl=40, ww=400)
Num 7	Windowing preset Lungs for CT images (wl=-400, ww=1500)

I	Invert
V	Show intensity value
Space	Stop/Play cine or video or multiframe
F	Fusion
Right click	Opens context menu
Up Arrow	Go to previous image
Down Arrow	Go to next image
Left Arrow	Go to previous series
Right Arrow	Go to next series
Ctrl + Shift + S	Activate fast scroll between series
Ctrl + K	Adding instance to the Key Object selection
Ctrl + Shift + K	Open the Key Object Selection management dialog
Shift + Z	Quick save KO, PR
P	Print active image*
Alt + V	Apply next Hanging Protocol
Alt + C	Apply previous Hanging Protocol
